# Saliva Self-Collection Kit Instructions

## IMPORTANT REMINDERS:

- Test must be registered online.
- You must follow observed collection instructions for specimen to be valid.
- Do not eat, drink, smoke or chew gum within 30 minutes of the collection.
- Specimens must be shipped same day you collect (Monday-Friday)

Before performing collection, confirm the nearest UPS drop off location and hours of operation. Ship the sample on the same day as collection before the last pick up or your sample may not be processed by the laboratory.


You must register your kit at [testinabag.com](http://testinabag.com) before you collect. Do not register before you are ready to collect as you will be asked to document the date and time of collection. Scan the QR code on the enclosed label to be taken directly to the registration page.


When ready to begin the collection, call 844-512-1385. Our representatives will walk you through the observed collection process. Your collection must be observed in order to be processed.

## User Instructions **YOUR COLLECTION MUST BE OBSERVED. DO NOT BEGIN UNTIL PROMPTED BY REPRESENTATIVE @ 844-512-1385**


Most people take between 2 and 5 minutes to deliver a sample following steps 1 to 5.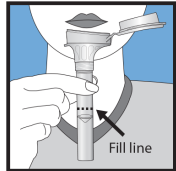


- 1 Spitting into funnel until the amount of liquid (not bubbles) reaches the fill line shown in picture #1.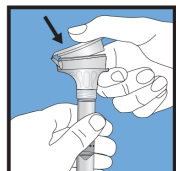


- 2 Hold the tube upright with one hand. Close the funnel lid with the other hand by pushing the lid until you hear a loud click. The liquid in the lid will be released into the tube to mix with the sample. Make sure that the lid is closed tightly.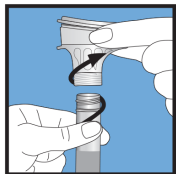

- 3 Hold the tube upright. Unscrew the funnel from the tube.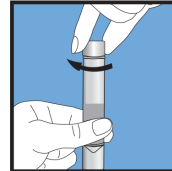

- 4 Use the small cap to close the tube tightly.

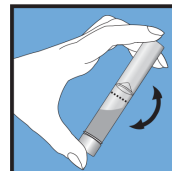- 5 Shake the capped tube for 10 seconds. Discard or recycle the funnel.

Remove QR code sticker from enclosed label and place around sealed tube.

Place the collection tube in the biohazard bag. Seal the biohazard bag by closing the zip lock seal. Place biohazard bag in return shipping bag and seal shut.

Immediately deliver package to a UPS drop box using the prepaid, pre-addressed return package. Ship the sample on the same day as collection before the last pick up or your sample may not be processed by the laboratory.

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